

# CUSTOM ULTRASONICS, INC.

## Model 20000

### Endoscope Storage Cabinet



INSTALLATION REQUIREMENTS:  
(by others)

ELECTRICAL: Dedicated 120 VAC 15 - 20 AMP 60Hz 1Ph  
Make certain the cabinet's electrical plug is inserted in a properly grounded, 120 VAC 15 - 20 AMP 60Hz 1Ph receptacle; otherwise electrical shock and injury could occur to the installer or damage could occur to the equipment.

#### MOUNTING:

Locate cabinet flush to desired wall and mount, using the mounting holes provided, with materials suitable to facility's structure and local building codes. Allow a 1" (min) clearance above the heat vent output on the top of the unit.

Make certain to have a minimum of (2) persons to support and mount the cabinet or bodily injury and physical damage to the equipment and surrounding area may occur.

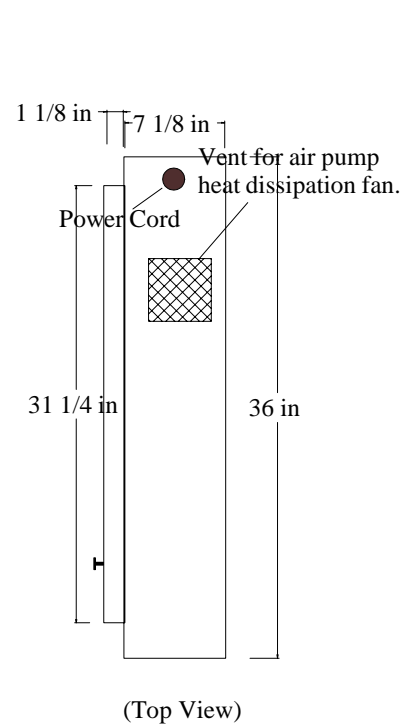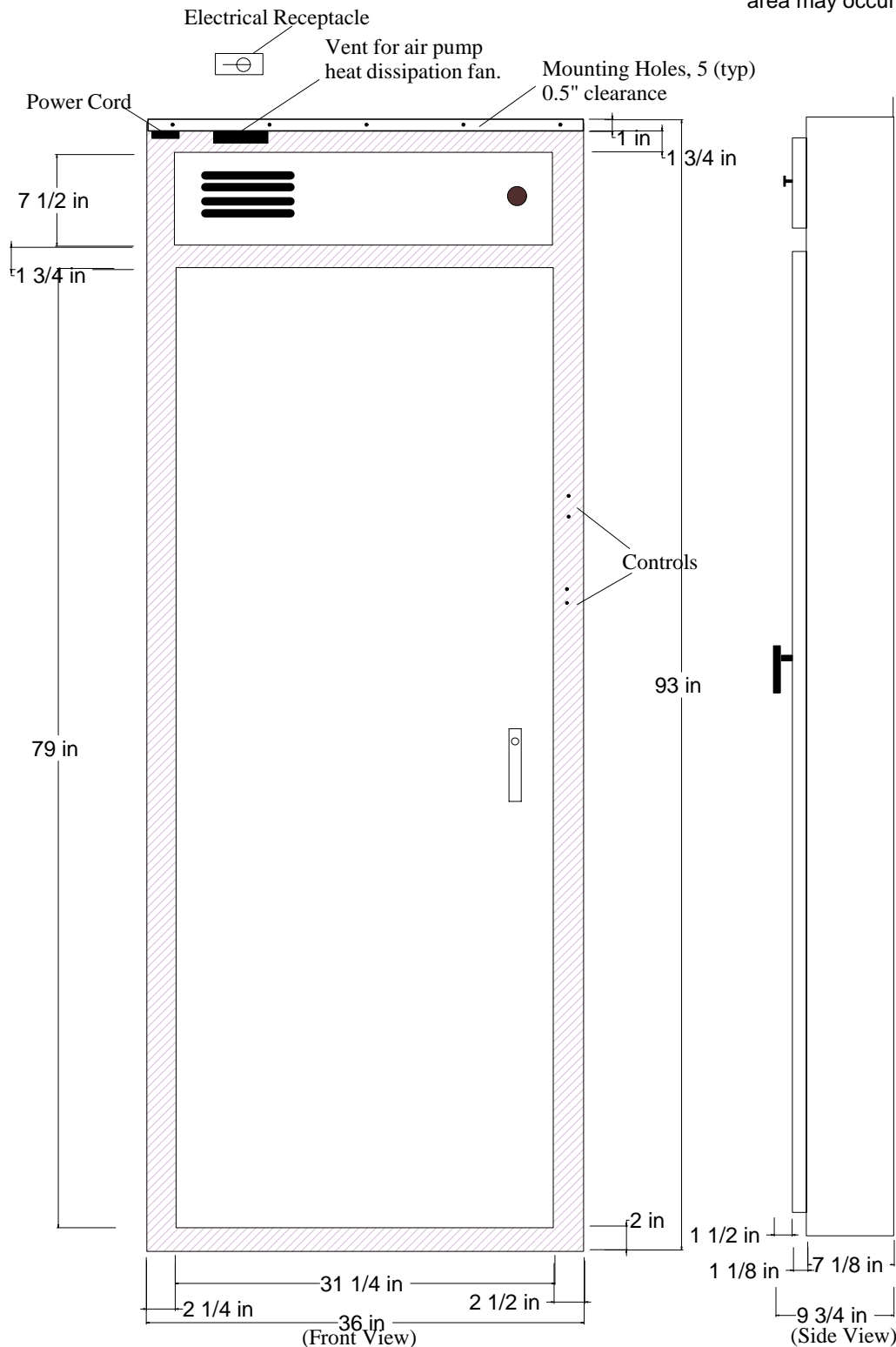

Weight ~ 175 lbs.

For information call  
(215) 364-1477  
(CompuCraft)

CS-20000-101, Rev C